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Dear Doctor,

Thank you for purchasing our **CORAL®**, professional dental stool.

Please do not hesitate to contact us at any time for further information or explanations about the equipment.

Please note that, in order to ensure that the unit is used correctly, you must read this manual carefully before setting the unit to work. Our equipment is designed in compliance with current safety standards and does not constitute a hazard for the user, provided that all the instructions are observed.

In wishing you all the best in your work, may we remind you that the reproduction of this manual in any form is strictly prohibited, and that the technical features of the unit may be modified at any time and without prior notice, in line with technological developments.

**EURONDA S.p.A.**

• **INDEX** •

• **INTRODUCTION**

GENERAL NOTES ON DELIVERY	P. 3
GENERAL INSTRUCTIONS	P. 3
WARRANTY INFORMATION	P. 4

• **PART 1**

OVERALL DIMENSIONS	P. 5
TECHNICAL SPECIFICATIONS	P. 5

• **PART 2**

STOOL PACKAGING	P. 6
STOOL ASSEMBLY	P. 7
STOOL ADJUSTMENT	P. 8

• **PART 3**

DESCRIPTION	P. 9
-------------	------

• **PART 4**

ROUTINE MAINTENANCE	P. 10
STOOL CLEANING	P. 10
SPECIAL MAINTENANCE	P. 10
CUSHION REPLACEMENT	P. 10
BACKREST REPLACEMENT	P. 10
CASTOR REPLACEMENT	P. 11
GAS CYLINDER REPLACEMENT	P. 11

• **PART 5**

EXPLODED DRAWING OF STOOL	P. 12
LIST OF COMPONENTS	P. 13
SCRAPPING INSTRUCTIONS	P. 14

## GENERAL NOTES ON DELIVERY

On delivery of the stool, check the condition of the packaging (conserve for further shipments).

Remove the packaging and check the following:

- that equipment corresponds to technical specifications;
- that there is no evident damage

In the event of damage or missing parts, inform the shipping agent, dental depot or Euronda S.p.A. immediately.

All dimensions specified in the present manual are not to be considered binding.

Drawings and any other documentation supplied with the equipment are property of Euronda S.p.A.; all rights reserved; distribution to third parties is strictly prohibited.

Partial or total reproduction of texts or illustrations is strictly forbidden.

## GENERAL INSTRUCTIONS

The present manual has been drawn up to provide instructions regarding:

- correct assembly
- operation
- routine maintenance

The equipment must be used in compliance with the instructions in this manual and exclusively for its intended use.

The user is legally responsible for installation, operation and use of the equipment.

The manufacturer shall not be held responsible for breakage, damage, or faulty operation of the equipment in the event of incorrect use or failure to carry out maintenance.

Intended use: this manual constitutes an integral part of the product and must be kept in an accessible place near the unit where it can be consulted easily and quickly. The unit is designed for use by professional, qualified persons only. The unit must be used for its intended purpose only.

**CORAL: stool for dental operator, professional use.**

## GUARANTEE INFORMATION

Euronda guarantees the quality of its equipment, if used in accordance with the instructions supplied in this manual, and according to the conditions printed on the GUARANTEE CERTIFICATE.


**WARNING:** the CUSTOMER USER must fill in all parts of the DETACHABLE SLIP of the guarantee certificate and send it to EURONDA S.p.A.

The guarantee is effective from the delivery date of the unit to the customer, which is confirmed by the return of the guarantee slip correctly filled in and signed. In the case of dispute, the date indicated on the purchase invoice, which shows the serial number of the unit, will be considered valid.

**NOTE:** Please retain the original packaging and use it each time the unit needs to be transported. The use of different packaging could damage the product during transportation.


## PART 1

## OVERALL DIMENSIONS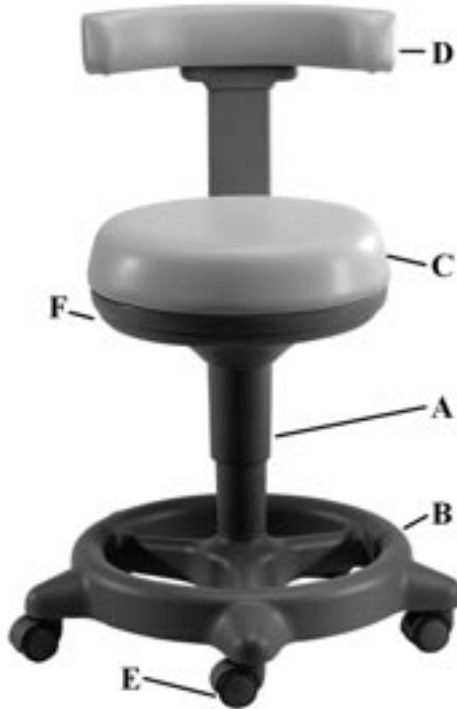

## TECHNICAL SPECIFICATIONS

<b>Adjustable height stool</b>	Min : 50 cm – Max: 68 cm (A)
<b>Base diameter</b>	Outside 60 cm – inside 46 cm (B)
<b>Seat</b>	> 390 x 90 mm (C)
<b>Backrest with adjustable position and depth (D)</b>	Depth max : 42,5 cm Depth min : 39,5 cm
<b>Wheels</b>	5 self-locking (E)
<b>Weight</b>	13 Kg
<b>Colour nylon structure</b>	RAL 7005
<b>Sky colour</b>	Several colour scales
<b>Overall dimensions (packed)</b>	610 x 630 x 530 mm
<b>Integrated operative system, adjustable at any point around circumference (F)</b>	

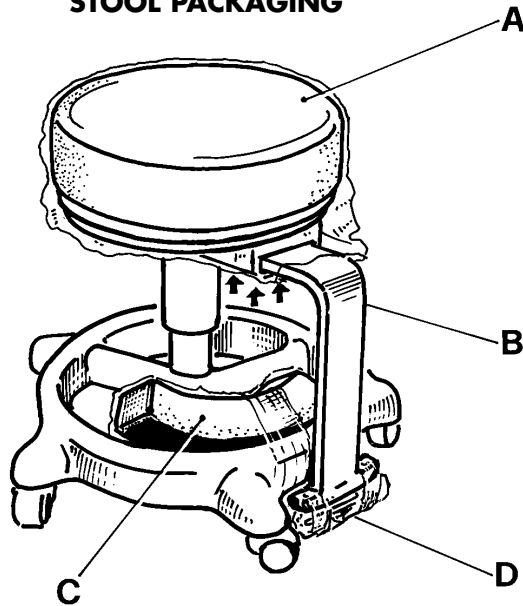**PART 2****STOOL PACKAGING**

After unpacking the stool, check that the structure and artificial leather comply with order specifications.

**The stool is delivered as follows:**

- seat-base assembly (A)
- backrest support inverted (B)
- anatomic backrest fixed to base (C)
- polyethylene bag (D) fixed to backrest support and containing one 6 mm. Allen key and two socket screws (M 8x20).
- plastic bag containing 5 wheels complete of pivot pin.


**Prior to assembly, proceed as follows:**

**ATTENTION: All maintenance must be performed by authorised qualified personnel.**

1. Open the polyethylene bag (D) and remove the Allen key and screws.
2. Use the Allen key to slacken the three screws fixing the backrest support (B) as indicated by the arrows.
3. Detach the anatomic backrest (C) from the base.
4. Insert the wheels on the opposite places under the base and press them.

**Proceed with assembly as described below.**


## STOOL ASSEMBLY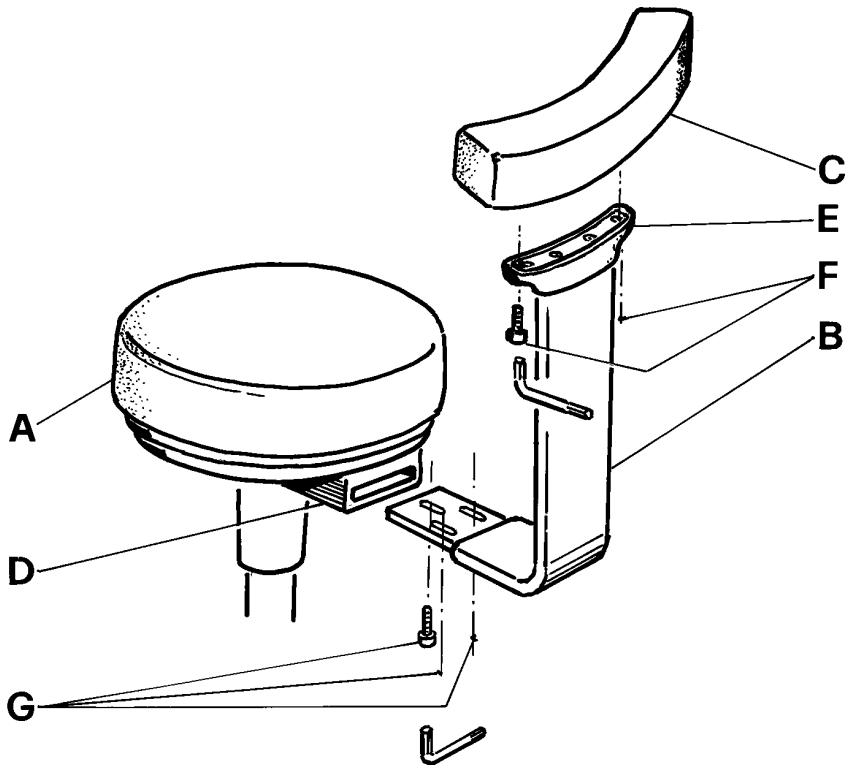

1. Fix the backrest support (B) to block (D) by means of the three screws (G), using the allen key supplied.
2. Fix the anatomic backrest (C) to bracket (E) by means of the two screws (F), using the allen key supplied.

The section "Stool adjustment" describes procedures for optimal seat adjustment.

**STOOL ADJUSTMENT**

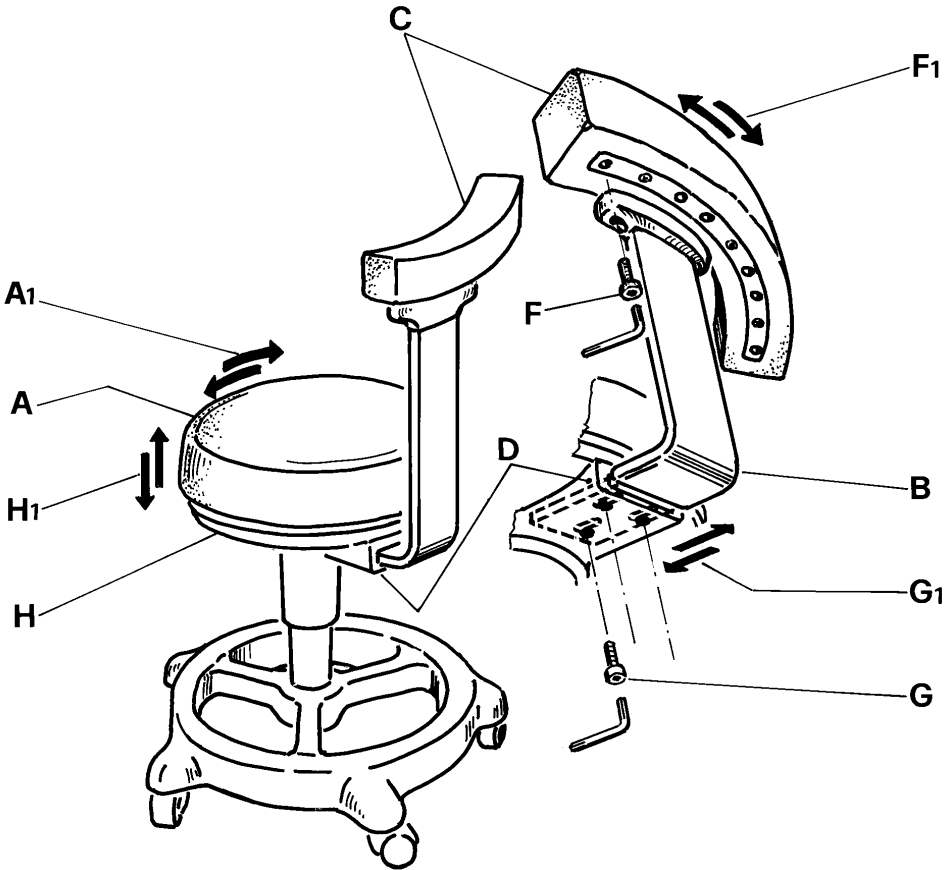1. The anatomic backrest (C) can be adjusted in the direction (F1) with respect to the support (B) in three positions: central, right side and left side.  
To adjust, remove screws (F) by means of the 6 mm Allen key, position the backrest as required, and tighten screws (F) to secure.
2. To adjust the depth (G1) of the backrest support (B) , slacken the three screws (G) by means of the 6 mm Allen key and slide the support (B) along the slots to the required position. To secure, tighten the three screws (G).
3. Seat (A) can be rotated in both directions (A1).
4. To adjust the seat height (H1), lightly press ring (H) under the seat, at any point around the circumference as required.

## PART 3

### DESCRIPTION

**Coral, professional dental stool, manufactured in compliance with ISO 7493 European standards.**

Coral, is the result of Euronda's long-standing experience in the construction of dentist stools and not only completely satisfies global requirements of new standards, but also features utmost reliability in terms of materials used, functionality, user comfort and structure ergonomics.

Salient features:

- Solid recyclable plastic structure, flame-retardant, non-scratch and impact resistant fitted with 5 self-locking wheels.
- Integrated operative system in lower section adjustable at any point around the circumference and protected by a special cover.
- Solid contoured backrest with three-point adjustment.
- Round cushion, non-deformable in high density layered foam, enabling optimal work position to minimise stress on the back, pelvis and legs.
- Smooth and rounded surfaces for enhanced hygiene and easy cleaning.

Coral,, in compliance with new standards, represents the ideal stool and intelligent choice for professionals.

Base structure

Seat and operative system

## PART 4

### ROUTINE MAINTENANCE

#### STOOL CLEANING

All the external parts of the stool have to be regularly cleaned with a detergent based of non-ionic-cationic surfactants, inorganic salts, glycol.

### SPECIAL MAINTENANCE

**ATTENTION: ALL MAINTENANCE MUST BE PERFORMED BY AUTHORISED QUALIFIED PERSONNEL.**

#### CUSHION REPLACEMENT

**(reference exploded drawing page 12)**

- a) Turn the cushion over and use the 6 mm Allen key to slacken the three screws (34) fixing the backrest support (7) and remove the latter.
- b) Use a 4 mm Allen key to remove the three screws (33) fixing the hub cover (17) and lift to remove.
- c) Use a 4 mm Allen key to remove the three screws (30) fixing the fixed hub (8) to the padded cushion (1)
- d) Remove the lower section of the seat by lifting it from the base (23).
- e) Remove the eight screws (27) securing the cushion to the fixed nylon ring (2) by means of a Phillips screwdriver.
- f) Remove the fixed ring from the cushion and fit on the new cushion, taking care that the three slots on the ring are aligned with the three cushion fixing bushes.
- g) Reassemble the seat in reverse of the above.

#### BACKREST REPLACEMENT

**(reference exploded drawing page 12)**

- a) Turn the stool over and use the 6 mm Allen key to slacken the two screws (32) fixing the backrest (5) to the backrest support (7).
- b) Use a Phillips screwdriver to remove the three screws (31) securing the fixing plate (4) to the backrest (3).
- c) Remove the fixing plate and fit onto the new backrest with the aid of a rubber faced mallet.
- d) Reassemble in reverse order of the above.

## CASTOR REPLACEMENT

(reference exploded drawing page 12)


- a) Turn over the seat
- b) Pull the castor (26) hard to remove together with the pin.
- c) Insert the new castor with pin by pushing it hard into the seat.

## GAS CYLINDER REPLACEMENT

(reference exploded drawing page 12)

- a) Position the stool at minimum height and turn it over;
- b) Use the 6 mm Allen key to slacken the three screws (34) fixing the backrest support (7) to the hub (9);
- c) Withdraw the circlip (24) from under the base (23);
- d) Withdraw the base (23) from the gas cylinder (12);
- e) Remove the washers (19), bearing (20) and rubber rings (18) from the gas cylinder;
- f) Use a 4 mm Allen key to remove the three screws (33) and remove the hub cover (17) and tube cover (16)
- g) Use a 4 mm Allen key to remove the three screws (30) securing the fixed hub (8) to the cushion (1);
- h) Grip the gas cylinder firmly with one hand and with the other use a rubber faced mallet to strike the three arms of the fixed hub (8) until the gas cylinder (12) can be removed;  
N.B. Take care not to strike the three grub screws (29);
- i) Insert the new gas cylinder in the fixed hub (8), grip the pump after turning it over (the fixed hub facing upwards) and use a steel mallet to strike the hub until it fits correctly on the cylinder;
- j) Check that the three piston levers (13) do not move with respect to the hub; to eliminate play adjust the three grub screws (29) by means of a 2 mm Allen key;
- k) Reassemble in reverse order of the above.

### PART 5 EXPLODED DRAWING OF STOOL

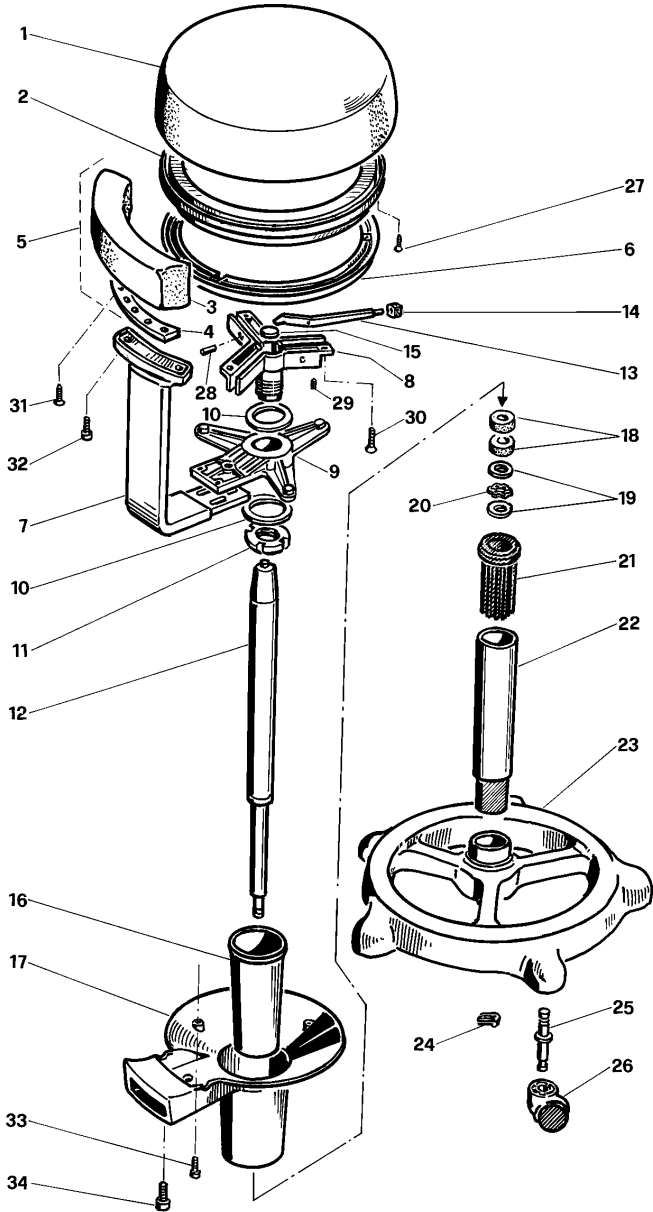English

## LIST OF COMPONENTS

Refer to the codes indicated below when ordering spare parts

NUMBER	QUANT.	PART	CODE
1	1	Cushion	918256
2	1	Fixed nylon ring	918508
3	1	Backrest	918267
4	1	Fixing plate	918016
5	1	Backrest with fixing plate and screws	918267/C
6	1	Nylon handle ring	918504
7	1	Nylon backrest support	918517/C
8	1	Fixed hub	918061
9	1	Mobile hub with nylon bush	918062/C
10	2	Nylon spacer	918005
11	1	Self-locking nut (diam. 45x15)	950070
12	1	Gas cylinder (h. 330 c. 200)	960100
13	3	Piston control lever	918003
14	3	Lever seal (13x9, diam. 5)	953073
15	1	Piston cover	918006
16	1	Nylon cylinder tube cover	918652
17	1	Nylon hub cover	918535
18	2	Rubber ring (diam. 20x10x8)	-
19	2	Washer (diam. 24x8x2)	-
20	1	Gas cylinder bearing	-
21	1	Nylon bush	-
22	1	Chrome plated piston guide tube with extension	918153/C
23	1	Nylon base	918526
24	1	Circlip	-
25	1	Wheel pin (positive locking; diam. 11)	-
26	1	Wheel - model AD38 grey nylon	979301
27	8	Self-tapping screw 3.5x16	993290
28	3	Spring dowel (diam. 5x16)	984159
29	3	Grub screw (P.P. M4x8)	951073
30	3	M6x18 bolt	990208
31	3	Self-tapping screw (6.3 x 40)	993300
32	2	Socket screw (M8x20)	991403
33	3	Socket screw (M5x12)	991295
34	3	Socket screw (M8x20)	991403

## SCRAPPING INSTRUCTIONS

The CORAL®, professional dental stool has been manufactured using ferrous materials and plastic materials. No particular precautions are required when demolishing the unit.

Do not dump the unit in an unguarded place; consign it to a waste disposal agency for scrapping. Always refer to the binding laws in the country of use when scrapping equipment.

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