# A 865

- de** Gebrauchsanweisung Adapter für die Aufbereitung von Kanülen von 3F und MF - Spritzen
- da** Brugsanvisning Adapter til genbehandling af kanyler på 3F- og MF-sprøjter
- en** Operating instructions Adapter for reprocessing canulae from 3F and MF syringes
- es** Instrucciones de manejo Adaptador para la preparación de cánulas de jeringuillas 3F y MF
- fi** Käyttöohje Sovitin 3F- ja MF-ruiskujen puustinkärkien koneelliseen käsittelyyn
- fr** Mode d'emploi Adaptateur pour le traitement des canules de seringues 3F et MF
- hr** Upute za uporabu Adapter za obradu kanila od 3F i MF – injekcija
- it** Istruzioni d'uso Adattatore per il trattamento di beccucci per siringhe 3F e MF
- nl** Gebruiksaanwijzing Adapter voor de behandeling van canules van 3F- en MF-spuiten
- no** Bruksanvisning adapter for klargjøring av kanyler til 3F og MF-sprøyter (multifunksjonssprøyter)
- pt** Instruções de utilização Adaptador para o reprocessamento de cânulas de seringas 3F e MF
- sl** Navodila za uporabo Adapter za pripravo kanil brizgalk 3F in MF
- cs** Návod k obsluze Adaptér pro přípravu kanyl stříkaček 3F a MF

**de** ..... 4  
**da** ..... 15  
**en** ..... 26  
**es** ..... 37  
**fi** ..... 48  
**fr** ..... 59  
**hr** ..... 70  
**it** ..... 81  
**nl** ..... 92  
**no** .....103  
**pt** .....114  
**sl** .....125  
**cs** .....136

---

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<b>Notes about these instructions</b> .....	26
Queries and technical problems.....	26
<b>Intended use</b> .....	27
<b>Items supplied</b> .....	28
Disposal of the packing material .....	28
<b>Warning and Safety instructions</b> .....	29
<b>Areas of application</b> .....	30
Installation .....	30
A 865 installation .....	30
Installing the holder with changeover mechanism .....	31
Preparing the load .....	33
Reprocessing .....	35
Reprocessing cannulae from 3F and MF syringes.....	35

### Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

### Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

### Additional information and comments

Additional information and comments are contained in a simple frame.

### Operating steps

Operating steps are indicated by a black square bullet point.

#### Example:

- Select an option using the arrow buttons and save your choice with *OK*.

### Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

#### Example:

Settings  menu.

### Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at [www.miele.com/professional](http://www.miele.com/professional).

The A 865 adapter is used to reprocess machine-reprocessible canulae from 3F and MF dental syringes in a washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical device manufacturer on how to reprocess their items by machine.


The adapter can be used in the following baskets and modules:

A 105/1

A 105

A 315

The washer-disinfector is generally referred to as “the machine” in these operating instructions. Reprocessible medical devices are referred to as “load items” if they are not more closely defined.

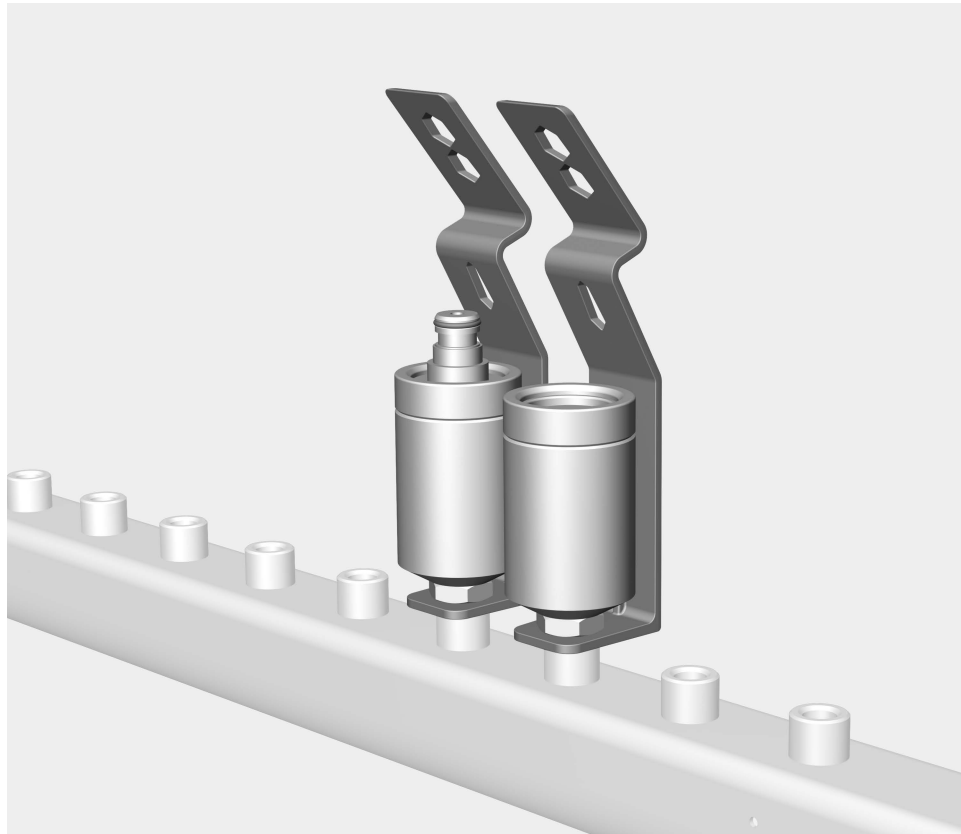Holder for reprocessable cannulae from 3F and MF syringes A 865, height approx. 122 mm, width approx. 32 mm, depth approx. 28 mm

### **Disposal of the packing material**

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

## en - Warning and Safety instructions

---

Read the operating instructions carefully before using this holder. This prevents both personal injury and damage to the rinse fitting. Keep these operating instructions in a safe place.


 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The adapter has been approved for the applications stated in these operating instructions. Applications that go beyond those listed and use of the adapter with additional nozzles, irrigation sleeves and rinse fittings from the Miele range of accessories are described in the operating instructions for the accessories or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- ▶ Inspect all mobile units, baskets, modules and inserts daily as described in the “Maintenance” section in the Operating instructions for the washer-disinfector.
- ▶ For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments and unloading the machine. To avoid recontamination, wear clean gloves when unloading the machine.
- ▶ To clean and disinfect lumened instruments thoroughly from the inside, they must be attached to the relevant holders or nozzles on the injector manifolds. The connectors must not come loose during a programme sequence.  
At the end of every programme sequence, check that the connectors have not come loose.
- ▶ The last rinse, as a minimum, must be carried out with DI water.

### Installation

**A 865 installation** A 865 adapters are required to reprocess the reprocessable cannulae from 3F and MF syringes used in dental applications. The adapters are compatible for use with standard 3F and MF syringes made by manufacturers such as KaVo, Luzzani and Sirona.

- Screw the A 865 adapters into the connections on the injector manifold.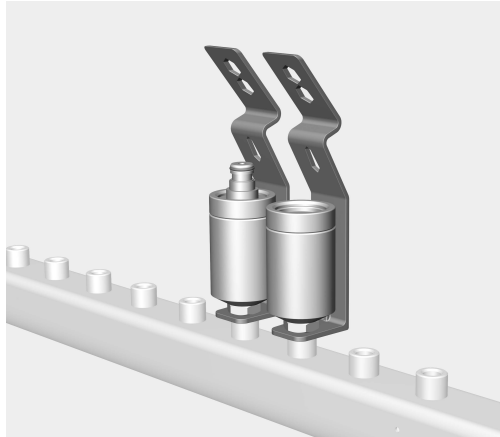


When A 865 adapters are used, the wash water must be filtered. The injector manifold must be fitted with an A 800 tubular filter. The wash water consists of the inflowing water, the process chemicals and the soiling. The tubular filter is supplied with its own operating instructions. Follow in particular the instructions on how to perform regular cleaning of the tubular filter.

### Installing the holder with changeover mechanism

Depending on the geometry of the 3F and MF syringes, the holder ② (highlighted in grey) can be installed rotated through 180° (changeover mechanism).

- Place the compression spring ④ inside the sleeve ⑤.
- Push the holder ② onto the compression spring ④ and fit the lid ①.
- Screw the hex nut ⑦ onto the sleeve ⑤ as far as the stop.
- Connect the spring clip ⑥ to the thread of the sleeve ⑤.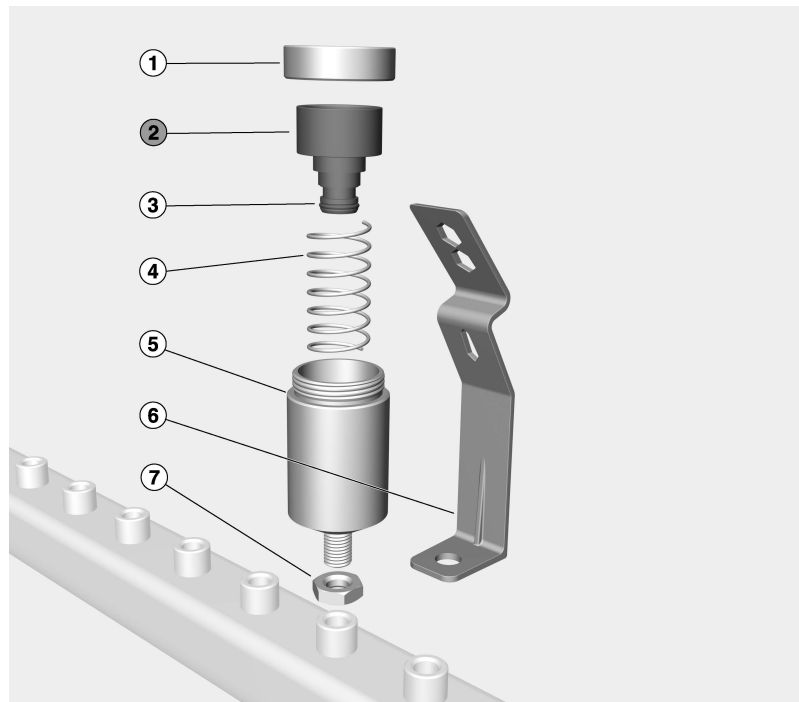


Holder facing downwards (changeover mechanism)

- ① Lid
- ② Holder
- ③ O-ring
- ④ Compression spring
- ⑤ Sleeve
- ⑥ Spring clip
- ⑦ Hex nut

## en - Areas of application

---

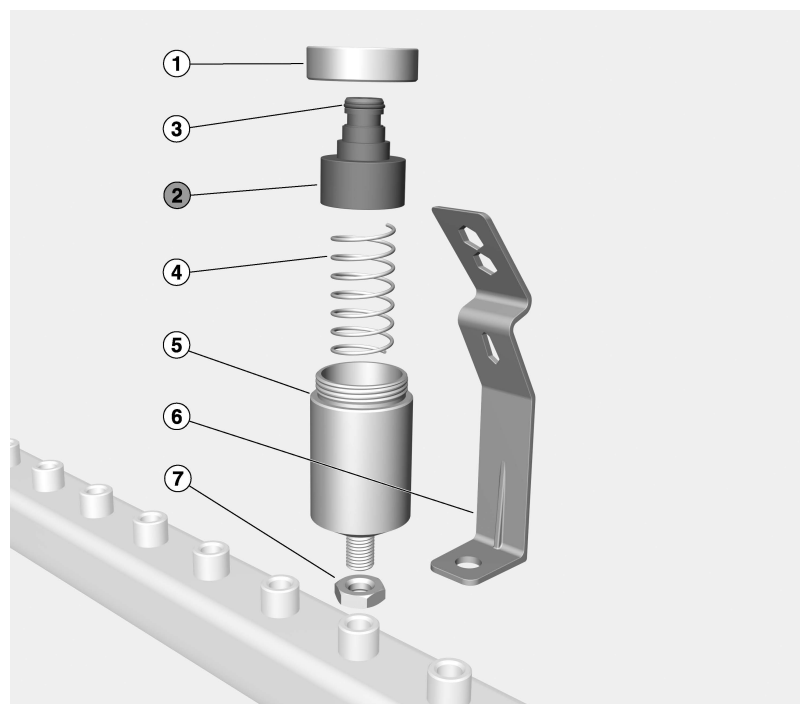
Holder facing upwards (changeover mechanism)

- ① Lid
- ② Holder
- ③ O-ring
- ④ Compression spring
- ⑤ Sleeve
- ⑥ Spring clip
- ⑦ Hex nut

The holder ② will not come out of the sleeve ⑤.  
Damage to the component caused by incorrect handling.  
Unscrew the adapter from the injector manifold and turn through 180°. The holder ② will come out of the sleeve ⑤.

## Preparing the load

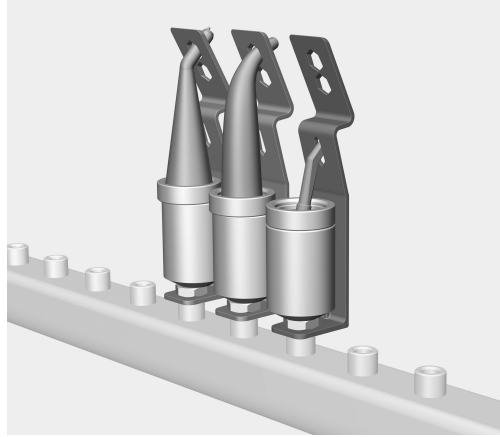
There are 3 recesses in the spring clip for cannulae of different sizes from 3F and MF syringes.

- Insert the cannulae from the 3F and MF syringes into the holder on the adapter.
- Position the cannulae from the 3F and MF syringes in the recess in the spring clip.

⚠ High spring force of compression spring



The spring force can cause the cannulae from the 3F and MF syringes to jump out of the holder.

Load the cannulae from the 3F and MF syringes then press down on them and use the spring force to slowly position the cannulae into the appropriate recesses on the spring clips.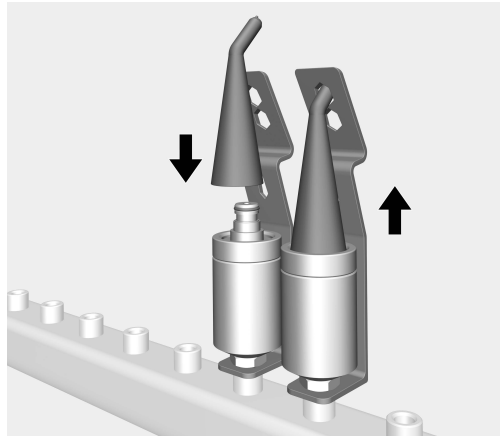

## en - Areas of application

---

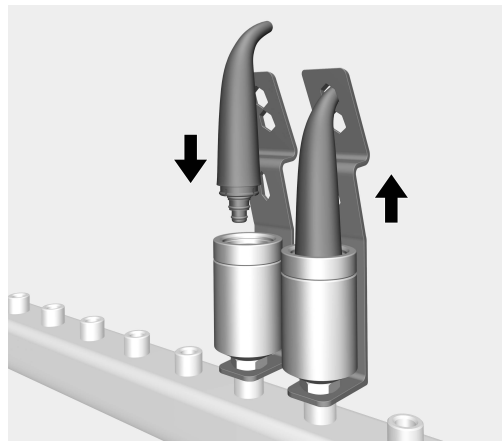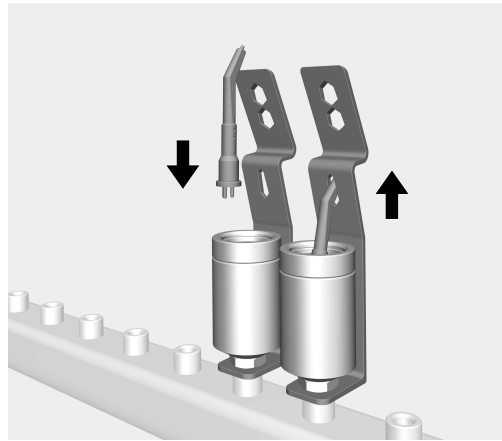- To remove the cannulae from the 3F and MF syringes after reprocessing, press them down into the holder on the adapter.
- Pull the cannulae from the 3F and MF syringes out of the spring clips and then take them out of the holders.

**⚠ High spring force of compression spring**

The spring force can cause the cannulae from the 3F and MF syringes to jump out of the holder.

To unload the cannulae from the 3F and MF syringes, press down on them and use the spring force to slowly pull the cannulae out of the spring clips and then remove them from the holders.

## Reprocessing

For validation purposes please make sure that you follow the loading instructions given on the template.

⚠ Risk of infection due to insufficient disinfection.

Load items that become detached from the irrigation connector during reprocessing will not be sufficiently disinfected on the inside.

Any items that become detached from the irrigation connector during reprocessing must be reprocessed again.

### Reprocessing cannulae from 3F and MF syringes

- Check that the cannulae from the 3F and MF syringes are free of blockages.

Before using cannulae from 3F and MF syringes again following reprocessing, a functional check must be carried out, e.g. spraying into a basin to ensure lumens are clear. The functional check must not be carried out until shortly before the 3F and MF syringes are due to be used on the patient.

⚠ Risk of injury caused by load.

There is a risk of injury when loading and unloading due to possible sharp edges, rims or pointed ends.

To minimise the risk of injury, loading should take place from the back forwards and unloading should take place in the reverse order.

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