DEXIS Titanium

by **KAVO**

QUICK START GUIDE
GUIDE DE DÉMARRAGE RAPIDE
KURZANLEITUNG

WELCOME

Congratulations on your investment in the DEXIS™ Titanium by KaVo!

A smart sensor that delivers market leading performance providing confidence, assurance, and next generation innovation for Dental Professionals seeking the best in digital imaging.

TABLE OF CONTENTS

Supported Software	4
Easy Sensor Installation in DEXIS software	5
Sensor Kit Contents	6
NEW Gesture Initiated Series.....	8
NEW Smart Sensor Positioning	9
NEW Smart Hole Adaption Holder System	10
Getting Oriented to the NEW Holder System.....	14
Universal Hand-Held Adaptor	13
Hygienic Barriers	14
Sensor Sanitation.....	15

EASY INSTALLATION

Warning: Always ensure supported Software is already installed prior to installation of your sensor.

1 SUPPORTED SOFTWARE INSTALLED
DEXIS Titanium operates directly in the following software environments:

- DEXIS 9.5.1 or Higher
- DEXIS 10.2.0 or Higher

2 Loading the software driver update that is in your kit
Insert the USB and simply install your DEXIS Titanium Drivers

For supported hardware configurations see sensor manual for minimum system requirements.

3 Plug in your sensor


4 Automatically self loads within 2 minutes!

DEXIS TITANIUM SENSOR KIT

1	DEXIS Titanium Sensor - Direct USB intra-oral digital X-ray sensor	(1.012.6180)
2	DEXIS Titanium USB Software Kit	(1.013.3134)
3	Sensor Garage - To store and protect sensor when not in use	(1.013.3841)
4	USB Extender, 3 Ft. (1 m) For easy access to computer's USB port	(1.007.3046)
5	Sensor Sheaths Sample Pack (100) - Hygienic barrier for the sensor, bag of 100	(1.013.3253)
-	DEXIS Titanium Sensor - User Manual (not shown)	

Top tier contents

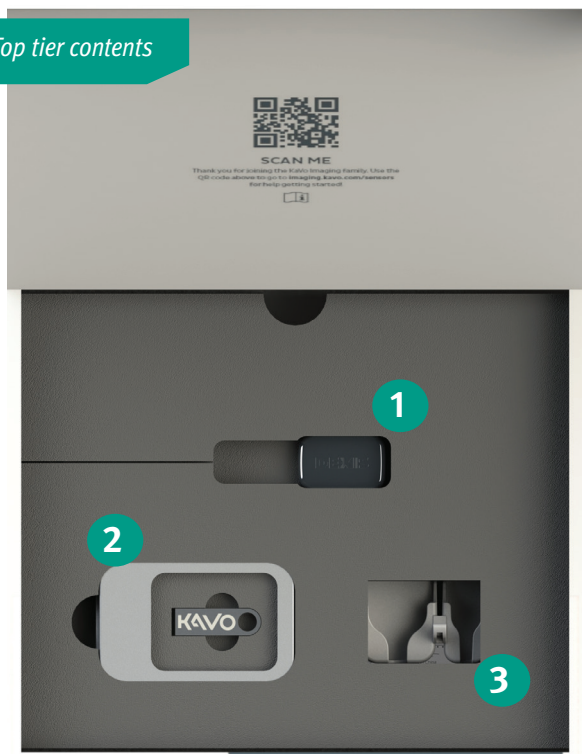
Bottom tier contents


DEXIS TITANIUM ACCESSORY KIT

6	Horizontal Bitewing Holders (2)	(1.013.1099)
7	Vertical Bitewing Holders (2)	(1.013.1100)
8	Anterior Holders (2)	(1.013.1098)
9	Posterior Holders (2)	(1.013.1097)
10	Universal Alignment Bar (2)	(1.011.1451)
11	Universal Hand-held Adapter	(1.013.1101)
12	Cable Clip Set Used to attach sensor cable to the alignment bars - #10 and #13	(1.013.1114)
13	Bitewing Alignment Bar	(1.011.1450)
14	* Universal Alignment Ring	(1.010.3775)
-	DEXIS Titanium Holder - User Manual (not shown)	

Top tier contents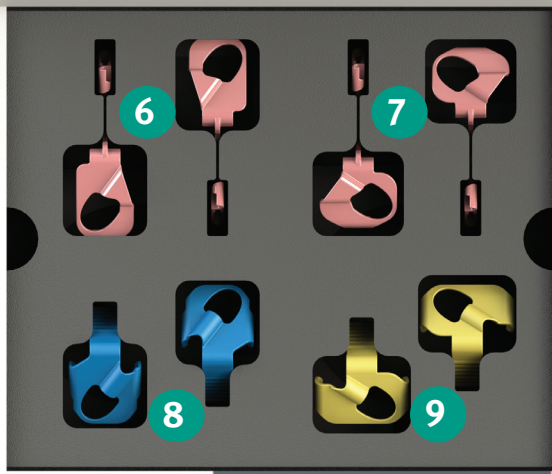

Bottom tier contents

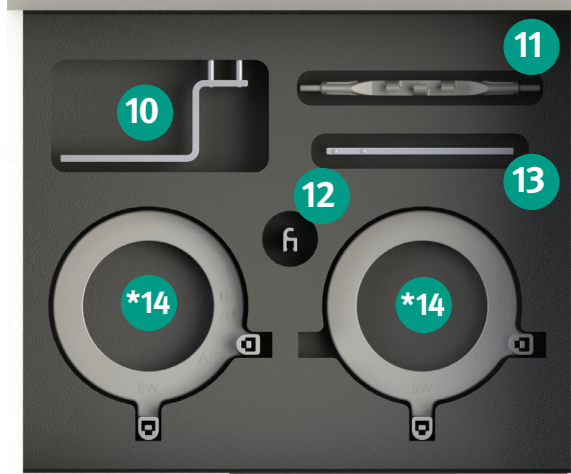* An alternative Universal Alignment Ring (1.008.1525 or 1.008.1526) may be supplied, depending on your country's requirements.

GESTURE INITIATED SERIES

Step 1: Hold the sensor assembly steady for 2 to 3 seconds

Vertical up and down motion for three cycles triggers **Vertical Bitewing Series**

Horizontal back and forth motion for three cycles triggers **Horizontal Bitewing Series**

Circle launches **Full Mouth Series**

To **Go Back** during the series use a triangle motion

To **Cancel** a Series, complete step 1, then move the sensor in a “C” motion

Complete use of gesture initiated series is referenced in Section 1.5 of the sensor manual.


NEW SMART SENSOR POSITIONING SYSTEM

Vertical
Bitewing

Horizontal
Bitewing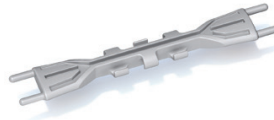

Universal Holder
Adaptor

Garage

Horizontal
Periapical

Vertical
Periapical

Universal Ring

Bitewing Bar

Universal PA Bar

Accessories sold separately

Cable Clips


Hand Held


Correct placement of the holder system is outlined in Section 3 of the sensor manual.

NEW SMART HOLE ADAPTION HOLDER SYSTEM


The NEW holder hole system is designed to support our custom DEXIS bars and other bars that may exist in offices today.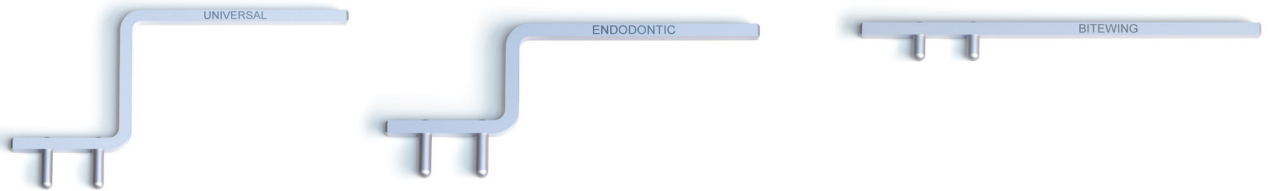

The holes facing the back of the NEW DEXIS Titanium holder system supports the DEXIS Titanium holder bars.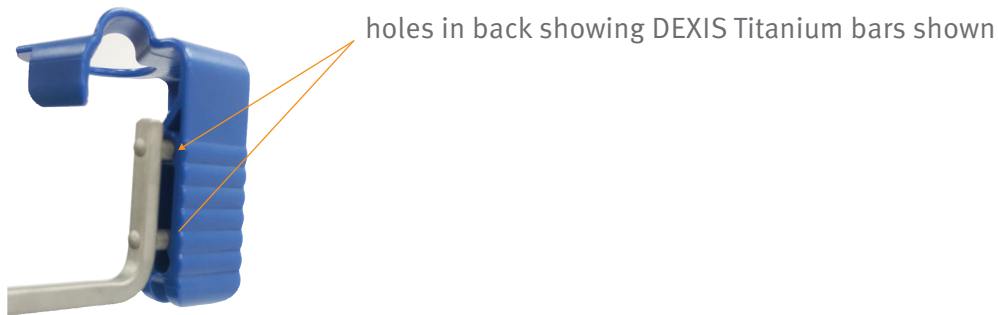

The holes that correspond to the face of the sensor on the NEW DEXIS Titanium holder system support major 3rd party bars.

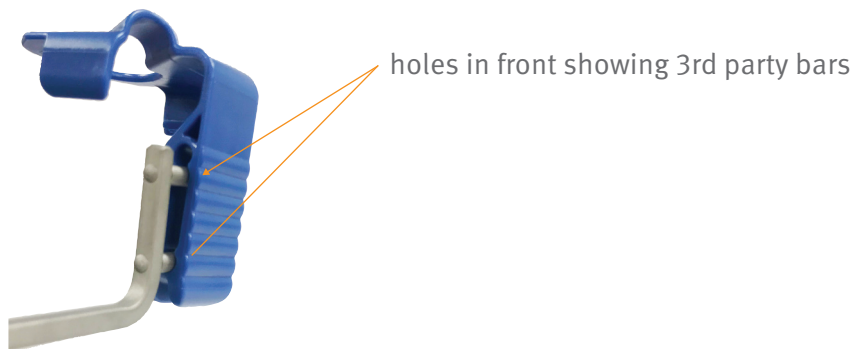GETTING ORIENTED TO THE NEW HOLDER SYSTEM

Vertical Bitewing holder and assembly

Horizontal Bitewing holder and assembly

THE NEW HOLDER SYSTEM CONTINUED


Horizontal Periapical holder and assembly

Vertical Periapical holder and assembly

UNIVERSAL HAND-HELD ADAPTER
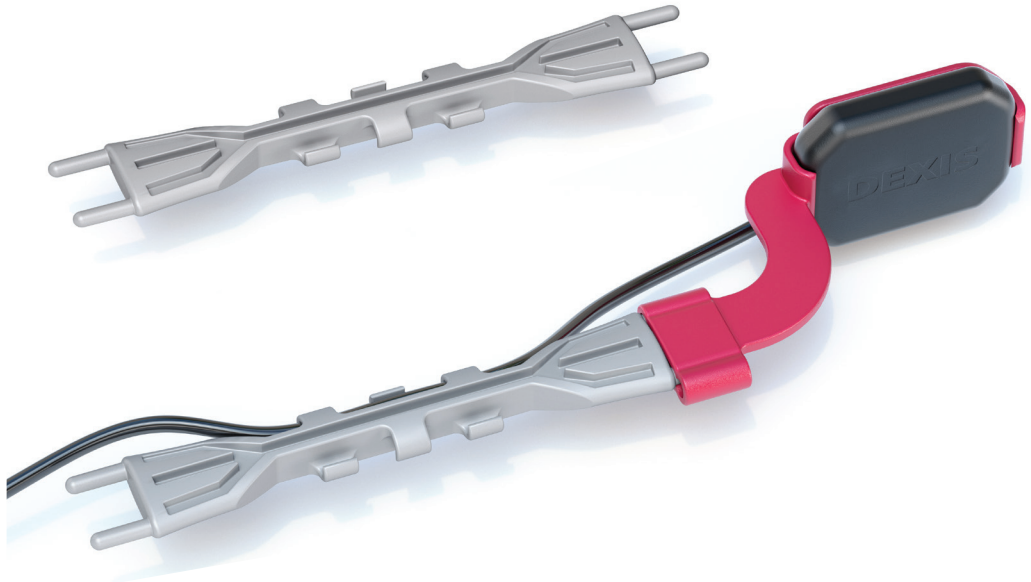


Universal Holder Adaptor and Assembly

HYGIENIC BARRIERS

Over Sensor barrier

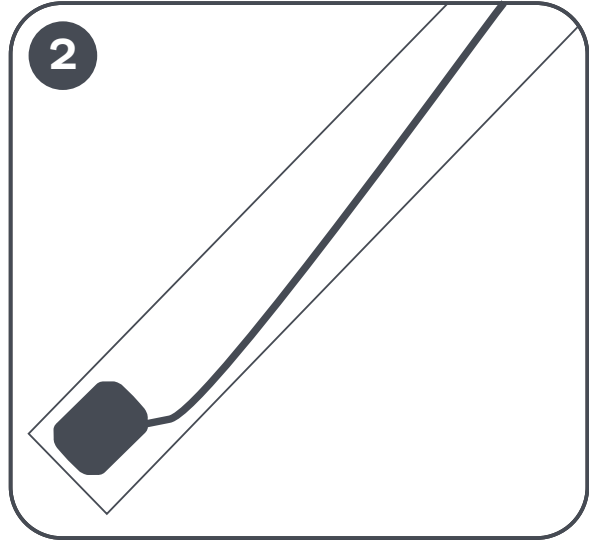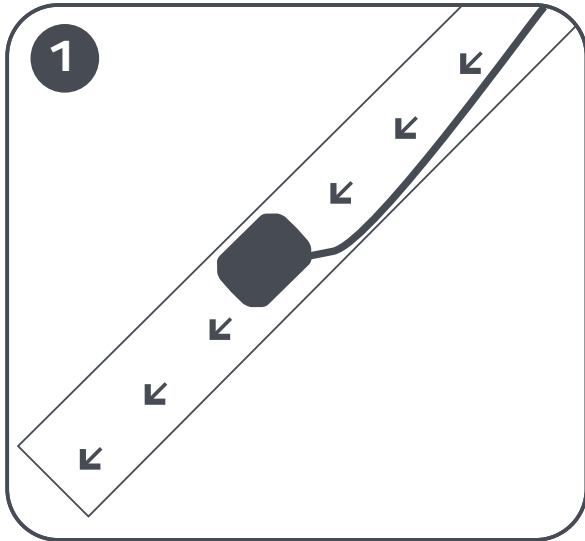SANITATION

Complete sensor care instructions are included in Section 2.7 of the sensor manual.

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Dental Imaging Technologies Corporation
PaloDEx Group Oy