**NAKANISHI INC.** 

[www.nsk-inc.com](http://www.nsk-inc.com)

700 Shimohinata Kanuma-shi  
Tochigi 322-8666,  
Japan

**NSK Dental Spain SA**

[www.nsk-spain.es](http://www.nsk-spain.es)

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**NSK Oceania Pty Ltd**

[www.nsk-inc.com](http://www.nsk-inc.com)

Unit 22, 198-222 Young St.  
Waterloo, Sydney,  
NSW 2017, Australia

**NSK France SAS**

[www.nsk.fr](http://www.nsk.fr)

19 avenue de Villiers  
75017 Paris,  
France

**NSK America Corp**

[www.nsk-inc.com](http://www.nsk-inc.com)

700 Cooper Court  
Schaumburg, IL 60173,  
USA

**NSK United Kingdom Ltd**

[www.nsk-uk.com](http://www.nsk-uk.com)

Office 5, Gateway1000,  
Arlington Business Park, Whittle Way,  
Stevenage, SG1 2FP, UK

**NSK Middle East**

[www.nsk-inc.com](http://www.nsk-inc.com)

Room 6EA-701, 7th Floor, East Wing No.6  
Dubai Airport Free Zone,  
PO Box 54316 Dubai, UAE

Specifications are subject to change without notice.

'09.00.03 ©

OPERATION MANUAL

**NSK**

 0197

Please read this Operation Manual carefully  
before use, and file for future reference

Contra Angled Handpiece

**S·Max**

S-Max MC

**MADE IN JAPAN**

OM-C0429E

Powerful Partners®

## Intended to use

The handpiece is designed only for clinical dental treatment use by qualified personnel.

### CAUTION

- When operating the handpiece always consider the safety of the patient.
- Check for vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using the handpiece immediately and contact your authorized dealer.
- Do not attempt to disassemble the handpiece or tamper with the mechanism.
- Do not connect or disconnect the bur and/or handpiece until the motor has completely stopped.
- Do not use bent, damaged or sub-standard burs. This may cause shank bend or brake even within the recommended speed.
- Always keep bur shanks clean. Dirt or debris in a chuck could cause poor bur concentricity or low chuck retention force.
- Do not use recommended bur and do not exceed the drive motor speed recommended the bur manufacturers.
- Always operate with water spray. Failure to do so may result in OVERHEATING.
- Do not wipe with, or clean or immerse in, high acid water or sterilizing solutions.

## CAUTION

- Perform periodical maintenance check.
- Do not overpressure to the handpiece and treatment area.

### M15L/M15,M25L/M25

Never perform PMTC (operation to give heavy torque by a low speed rotation) which it used rubber cap or brush. It may reason for the malfunction or stop rotation by less torque. Special Reduction Contra Angle Handpiece is required for the PMTC. However, if the micromotor has a function for the root canal explanation, you can perform it.

### M95L/M95

- Do not use long surgical burs. Do not use burs longer than 26mm.
- Entering foreign object into gear or inside of handpiece head may cause burn injury.
- Wear protection glass and dust mask for your health and safety.

Depressing the push-button while the handpiece is rotating will result in OVERHEATING of the handpiece head. Special caution must be exercised during use to keep cheek tissue AWAY from the push-button of the handpiece. Contact with cheek tissue may cause the push-button to depress and burn injury to the patient may occur.

- Even if it does irrigation during operation, bearing is worn and it may overheat or noise. Ask repair for your dealer.

## 1. Mounting/Removing the Handpiece and Motor

### 1) Mounting

Align the handpiece to the Motor and push together. For M15L,M25L,M95L, Locate the handpiece and alignment pin straight into positioning of the motor.

### 2) Removing


Hold the motor and the handpiece, and pull off carefully, in a straight line.

## 2. Connecting/ Disconnecting the Bur

### (1) Connecting

#### a) M15L,M15,M25L,M25 (Contra Angled Bur)

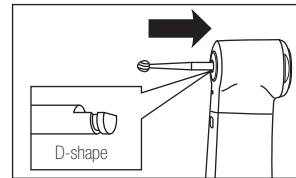- 1) Mount handpiece and Micromotor.
- 2) Insert the bur into the head and turn it slightly.(If bur and head in right position, bur is all the way seated in the head)
- 3) Insert the bur into the head until it click.(Fig.1)

\*M15L

Fig. 1


b) M95L, M95 (FG bur)

- 1) Mount handpiece and Micromotor.
- 2) Depressing the pushbutton and insert the bur into the chuck. (Fig. 2, , )
- 3) Make sure it is inserted firmly till the end of the head. Do not push the Push Button. (Fig. 2, )


### CAUTION

If the bur turns and it cannot match the D-shape area, disconnect the motor then connect it again. (Fig. 1)

- (2) Removing  
Release the button and pull the bur from the head. (Fig. 3)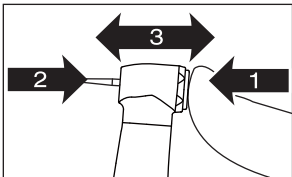

\*M95L Fig. 2

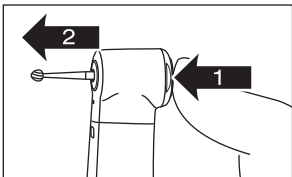\*M15L Fig. 3

## 3. Maintenance

Refer to the Maintenance Guide OM-Z0135E.

This handpiece can be cleaned and disinfected with a Thermo-Disinfector.


This handpiece can be autoclavable up to Max. 135°C.

## 4. Cleaning the Cellular Glass Optic Rod (M15L, M25L, M95L)

Wipe clean the Cellular Glass Optic Rod entry point and exit point with an alcohol-immersed cotton swab. Remove all debris and oil. (Fig. 4)

### CAUTION

Do not use a sharp tool to clean the Cellular Glass Optic Rod. It could damage the glass and reduce the light transmission. If illumination becomes dim contact your dealer.

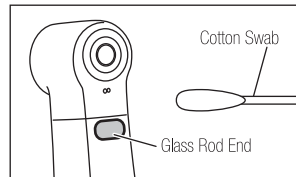\*M15L Fig. 4

## 5. Servicing

Exclusive part is necessary for cartridge exchange. If you need service for those handpieces, contact your dealer.

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## ENGLISH Specification

Model		M15L	M15	M25L	M25	M95L	M95
Gear Ratio		4:1 Reduction		1:1		1:5 Increase	
Max. Rotation Speed (Motor)		40,000 <sup>-1</sup> (rpm)					
Max. Rotation Speed (Bur)		10,000min <sup>-1</sup> (rpm)		40,000min <sup>-1</sup> (rpm)		200,000min <sup>-1</sup> (rpm)	
Irrigation		Internal Spray					
Spray		Single Water Spray				Quattro Water Spray	
Chuck		Push Button Chuck					
Light		Glass Rod Optic	-	Glass Rod Optic	-	Glass Rod Optic	-
B U R	Type	Ø2.35 Contra Angled Bur				Standard FG Bur	
	Shank Length	11.6mm				25mm (recommended 11.6mm)	
	Max. Dimantion	22.5mm				19.0mm	
	Work Dimantion	Ø4.0mm				Ø2.0mm	

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