Wirksamkeit

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- ◆ drying
- ◆ supporting the cleaning

WL-Blow

The compressed air gun WL-Blow with integrated air flow limiter is used to blow out, to dry and to support the cleaning of hollow part instruments and transmission instruments after cleaning and disinfection with WL-clean/WL-cid**. WL-Blow is intended for use in the processing room and can be connected to the existing oil-free dry air compressor with bacterial filter!

Field of application

Removing of residues of cleaning and disinfection agents and maintenance products in hollow part instruments and transmission instruments.

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REF 4158 WaterJet*

WaterJet

The rinsing device „WaterJet“ with integrated water flow limiter is used to pre-rinse and facilitates the cleaning before the further treatment of outer and inner surfaces of dental transmission instruments and general instruments. The rinsing device “WaterJet” is designed for the use in the preparation room and can be connected to the drinking water line.

Field of application

Facilitates the removing of organic residues on outer and inner surfaces of medical devices such as general instruments or dental transmission instruments.

Application

Immediately after the treatment of the patient flush the inside of the used instruments 3 x for 3 seconds and rinse them outside, if necessary repeat the procedure.

Application only by qualified personnel.

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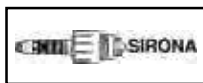
REF 3530

WL-Adapters

Universal adapters for the WL-series:

WL-Adapter 01** (plastics)

Universal adapter with Luer connector for transmission instruments (ISO 9168) for blowing-out of cleaning and maintenance products (WL-clean, WL-cid, maintenance oil) with WL-dry/WL-Blow. Also suitable for blowing out by using the multifunction syringe and to bring in the cleaning solution WL-clean via Luer connection e.g. into general hollow part instruments.

Adapters for spray water and spray air channels:

WL-Adapter 02** (stainless steel)

Adapter for spray water and spray air channels of all **contra-angles and hand pieces with ISO-interface.**

WL-Adapter 03** (stainless steel)

Adapter for spray water and spray air channels of **turbines with „Sirona“-interface.**

WL-Adapter 04** (stainless steel)

Adapter for spray water and spray air channels for **turbines with „Multiflex“- (KaVo/SciCan) interface.**

WL-Adapter 06** (stainless steel)

Adapter for spray water and spray air channels for **turbines with „Roto-Quick“-interface (W&H).**

WL-Adapter 07** (plastics)

Adapter for long shank contra-angles solely suitable for spray water and spray air channels of **Sirona T1 Classic-Line.**

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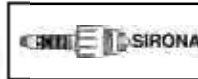
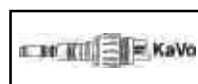
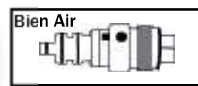
REF 3571

Adapter 1-U + U-1a Kombi* (stainless steel)

Combined adapter to clean and disinfect surgical contra-angles and hand pieces externally cooled with the WL-series. Consisting of the U-1 rinsing adapter for ultrasonic tips and U-1a screw attachment for hose plug connection. , see also page 29.

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(Adapter 02/G, 04/G) inkl. 1 Set Ersatz-O-Ringe

REF 3541

WL-Adapters

Adapters for complete internal cleaning:

WL-Adapter 02/G** (stainless steel)

Adapter for **contra-angles and hand pieces with ISO-interface** like adapter 02 but for complete internal cleaning/disinfection incl. head of instrument.

WL-Adapter 03/G** (stainless steel)

Adapter for **turbines with „Sirona“ interface** like adapter 03 but for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 04/G** (stainless steel)

Adapter for **turbines with „Multiflex“- (KaVo/SciCan) interface** like adapter 04 but for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 05/G** (stainless steel)

Adapter for **turbines with „Bien Air“-interface** for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 06/G** (stainless steel)

Adapter for **turbines with „Roto-Quick“-interface (W&H)** like adapter 06 but for complete internal cleaning with WL-clean/ WL-dry/ WL-Blow incl. head of turbine.

WL-Adapter 09/G** (stainless steel)

Adapter for **turbines with „NSK“-interface** for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 10/G** (stainless steel)

Adapter for **turbines with „Morita“-interface** for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 11/G** (stainless steel)

Adapter for **ACTEON-AIR N GO Airpolisher „Supra/Perio nozzle“** for complete internal cleaning/disinfection with WL-clean/WL-cid/ WL-dry/WL-Blow

Introductory set with 2 adapters

(adapter 02/G, 04/G) incl. 1 replacement set of o-rings

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REF 3575

WL-Adapters

Adapters for ultrasonic and aircaler tips

Adapter U-1 (stainless steel),

rinsing adapter for ultrasonic tips M 3 “internal thread” from EMS, Mectron, KaVo, Komet

Adapter U-2 (stainless steel)

rinsing adapter for ultrasonic tips M 3 “external thread” from EMS, Mectron, KaVo, Komet, NSK

Adapter U-3 (stainless steel)

rinsing adapter for ultrasonic tips M 3/a “internal thread” from Satelec, NSK

Adapter U-4-S (stainless steel)

rinsing adapter for ultrasonic tips in the-Piezosurgery “internal thread” from NSK (VarioSurg), Mectron (Piezosurgery)

Adapter U-5-S (stainless steel)

rinsing adapter for ultrasonic tips in the ultra-sonic surgery “external thread” from Sirona (Siroprep and Siroimplant)