**NSK**

Contra Angle Handpiece

**Ti·Max Z**

Z10L / Z10 / Z15L / Z15 / Z24L  
Z25L / Z25 / Z85L / Z85 / Z95L / Z95

OPERATION  
MANUAL

 0197 MADE IN JAPAN

OM-C0442E 002

## User and Intended Use

User: Qualified Professionals

Intended Use: Dental Treatment

### CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the Handpiece always consider the safety of the patient.
- Check for vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using the Handpiece immediately and contact dealer.
- Should the Handpiece function abnormally, cease operation immediately and return the Handpiece to dealer for repair.
- Depressing the push-button while the handpiece is rotating will result in OVERHEATING of the handpiece head. Special caution must be exercised during use to keep cheek tissue AWAY from the push-button of the handpiece. Contact with cheek tissue may cause the push-button to depress and burn injury to the patient may occur.
- Do not attempt to disassemble the handpiece or tamper with the mechanism.
- Do not allow any impact on to the Handpiece. Do not drop the Handpiece.
- Do not connect or disconnect the Handpiece or bur until the drive motor has completely stopped.

- Do not use bent, deformed or damaged bur. If these burs are used, it bends suddenly, or it breaks during rotation. Bur may become difficult to remove because latch groove is worn or it has swollen.
- Always keep bur shanks clean. Dirt or debris in a chuck could cause poor bur concentricity or low chuck retention force.
- Do not exceed the drive motor speed recommended the bur manufacturers.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.
- Before use always confirm that the bur is correctly seated and locked into place.
- Always use with adequate water supply and coolant air or damage to the tooth structure and overheating may occur.
- If a foreign material enter to the gear and inside of handpiece, it may become a cause of burn injuries.
- Use eye protectors, dust mask, for safety and health during treatment.
- Perform periodical maintenance checks.
- If the handpiece has not been used for a long period, check for noise, vibration, and overheating before use.
- Do not wipe nor immerse the handpiece acidic water or acidic solutions.
- User is responsible for operation, maintenance and safety.

Z10L,Z10,Z15L,Z15,Z24L,Z25L,Z25

NSK recommends the use of specified handpieces for polishing. If other types of handpieces are used, polishing paste could enter the handpiece head, causing the failure of the push button or burn injury due to the overheating of the handpiece head.

#### Z85,Z85L

Use short shank FG Bur for a mini head. If long bur has used, it may cause of the omission of early wear of a bearing, Bur will bend, and breakage.

#### Z95,Z95L

- Do not use burs longer than 25mm.

# 1. Specification

Model	Z10L	Z10	Z15L	Z15	Z24L	Z25L	Z25	Z85L	Z85	Z95L	Z95	
Max. Rotation Speed (Motor)	40,000min <sup>-1</sup>											
Max. Rotation Speed (Handpiece)	2,500min <sup>-1</sup>		10,000min <sup>-1</sup>		40,000min <sup>-1</sup>			200,000min <sup>-1</sup>				
Gear Ratio	16:1		4:1		1:1			1:5				
Bur	ISO 1797-1 Type1 Ø2.35mm CA Bur							ISO 1797-1 Type3 Short Shank FG Bur		ISO 1797-1 Type3 Ø1.60mm FG Bur		
Optic	Glass Rod	-	Glass Rod	-	Glass Rod		-	Glass Rod	-	Glass Rod	-	
Bur applied Length	12.7mm				11.3mm	12.7mm		10mm		10.9mm		
Max Bur Length	22.5mm							21mm		25mm		
Max Bur diameter	Ø4.0mm							Ø2.0mm				
Water Spray Type	Single									Quattro		
Use Environment	Temperature: 10 - 40°C (No Condensation), Humidity: 30 - 75%											
Transportation and Store Environment	Temperature: -10 - 50°C, Humidity: 10 - 85%, Atmospheric Pressure: 500 - 1,060hPa											

## 2. Connecting/Disconnecting Handpiece and Motor

### 2-1 Connecting

- 1) Align the handpiece and motor alignment pin.
- 2) Then push the handpiece directly into the motor in alignment with the pin.

### 2-2 Disconnecting

Hold the motor and the handpiece, and pull it out.

### **CAUTION**

Connect ONLY E type motor (ISO3964) and do not exceed maximum motor speed such as shown on the specification.

### 3. Check before treatment

Follow the check below, before use. If any abnormalities are found, stop using the handpiece immediately and contact the authorized NSK dealer.

- 1) Check that the Head Cap is firmly tightened.
- 2) Insert the Bur. (Do not exceed maximum rotation speed for any item.)
- 3) Check that there is no abnormal rotation, vibration, noise and overheating.
- 4) If there are no immediate apparent abnormalities, continue to rotate the handpiece for about 1 minute. Then check that there is no abnormal heating on the outer surface.
- 5) Operate it if there are not abnormalities above.

### 4. Mounting/ Removing the Bur

#### 4-1 Mounting the Bur

##### 4-1-1 Mounting the Bur (CA Bur) to Z10,Z10L,Z15,Z15L,Z24L,Z25,Z25L

- 1) Insert and lightly turn the bur until it engages with the latch mechanism. (Fig. 1)
- 2) Push the bur further inwards until it clicks into place.

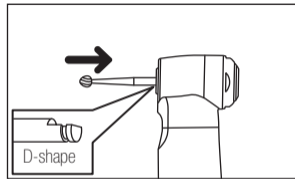
Fig. 1

#### 4-1-2 Mounting the Bur (FG Bur) to Z85,Z85L,Z95,Z95L

- 1) Connect handpiece and motor.
- 2) Depressing the push button and insert the bur into the chuck. (Fig. 2)

### CAUTION

- After the Bur is locked in place, lightly pull out the Bur to make sure the Bur is locked.
- If the Bur is not correctly secured it may release during operation and can cause injury.

#### 4-2 Removing the Bur (Fig.3)

- 1) Push the push button with a thumb.(①)
- 2) Pull out the bur.(②)

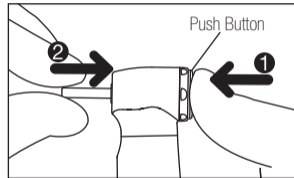
Fig. 2

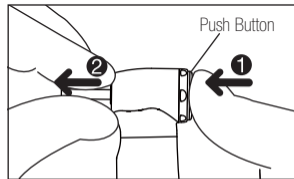
Fig. 3

## 5. Maintenance

After the treatment, be sure to perform the maintenance below.

### 5-1 Cleaning(Handpiece)

 This handpiece can be washed via Thermo Disinfector.

Scrub dirt and debris from the handpiece, and wipe clean with alcohol-immersed cotton swab or cloth. Do not use a wire brush.

\* Refer the manual of Thermo-Disinfector for performing it.

### 5-2 Cleaning(Z10L,Z15L,Z24L,Z25L,Z85L,Z95L)

Wipe clean the Cellular Glass Optic Rod entry point and exit point with an alcohol-immersed cotton swab. Remove all debris and oil.(Fig.4)

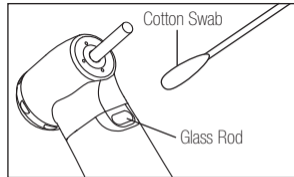
Fig. 4

## CAUTION

Do not use a pointed tool or sharp edged to clean the Cellular Glass Optic Rod. They may damage the mirror finish and reduce the light transmission efficiency. If Cellular Glass Optic Rod end has become damaged, return it to dealer for repair.

### 5-3 Lubirication and Sterilization

Refer to the Maintenance Guide **OM-Z0135E**.

## 6. Warranty

NSK warrants the handpiece against production errors and material defects. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the handpiece be not used correctly. Replacement parts are available for seven years beyond discontinuation of the model.

## 7. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

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**NSK Europe GmbH**
**NSK France SAS**
**NSK United Kingdom Ltd**
**NSK America Corp**
**NSK Oceania Pty Ltd**