RECIPROC® direct

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Directions for Use

Mode d'emploi

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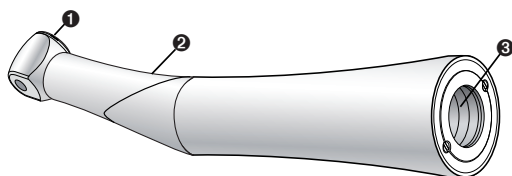
RECIPROC® direct

By opting for RECIPROC® direct, you have acquired a product that has been developed and tested with maximum care to meet the highest standards in terms of performance and handling.

Read these directions for use before using RECIPROC® direct for the first time. They explain how to handle your product and ensure trouble-free, economical and safe treatment. Please keep these directions for use in a safe place. The RECIPROC® direct directions for use are available in other languages upon request and on our website (www.vdw-dental.com).

1. Product Description

- ❶ Push button
- ❷ Handle
- ❸ Motor-side coupling as per ISO 3964 standard

2. Symbols Used

2.1 In these Directions for Use

Important!

Note

2.2. On the Product/Packaging

Manufacturer

Date of manufacture

Catalogue number

Serial number

Consult directions for use

CE marking

Can be subjected to thermal disinfection

Sterilisable up to the temperature indicated

RECIPROC®

3. Indications for Use

Endodontic contra-angle for power-driven root canal preparation on the patient using root canal instruments in reciprocation.

The contra-angle can be driven by pneumatic and electric motors with a coupling conforming to ISO 3964.

Non-intended use can damage the contra-angle and thus cause risks and hazards for patients, users and third parties.

4. Contraindications

None known.

Safety and effectiveness in pregnant women and children have not been established.

Only use original RECIPROC® instruments from VDW. Please refer to the information in the directions for use for RECIPROC® instruments.

5. Warnings

Read these warnings carefully before use. This section contains a description of potential side effects and safety risks for the product, users and patients.

- RECIPROC® direct may only be used by qualified persons in dental practices or clinics.
- RECIPROC® direct must be used in compliance with these directions for use.
- Repairs, alterations and modifications to the device are not permitted without obtaining the manufacturer's prior consent. VDW GmbH declines all responsibility if the device has been altered or modified. If a defect should occur, contact VDW or your VDW authorised service partner.
- Always use a rubber dam, otherwise the use of RECIPROC® direct may constitute a health hazard for the patient.
- Do not attach or detach the contra-angle during operation.
- Do not hold the contra-angle at eye level.
- Set the speed of **electric motors to 20000 rpm** (25000 rpm max.). With **motors driven by compressed air, work at maximum speed**. Working with settings other than the above mentioned RECIPROC® direct settings may constitute a health hazard for the patient.

 air-motor: max. speed e-motor: 20.000 rpm
--

6. Precautions

Read these precautions carefully before use. They enable you to use the product safely and aim to prevent injury to yourself and other individuals.

- Always make sure the operating conditions are appropriate.
- Before each use, check the contra-angle for damage and any loose parts (e.g. push button).
- Before each use, perform a trial run (see Section 8.4) and do not put the contra-angle into operation if there is any damage as described.
- Avoid contact between the push button and soft tissue (risk of burns caused by the push button becoming hot).
- Only use original RECIPROC® instruments from VDW and follow the instructions in their directions for use.
- Only insert the instrument when the contra-angle is stationary.
- Never touch the instrument while it is running or coming to a stop.
- The contra-angle does not have a torque limiter. Prepare the root canal carefully without exerting pressure on the contra-angle. If this rule is not observed, there is a risk of file breakage.
- Safety and effectiveness in pregnant women and children have not been established.

7. Adverse Reactions

None known.

8. Professional Use

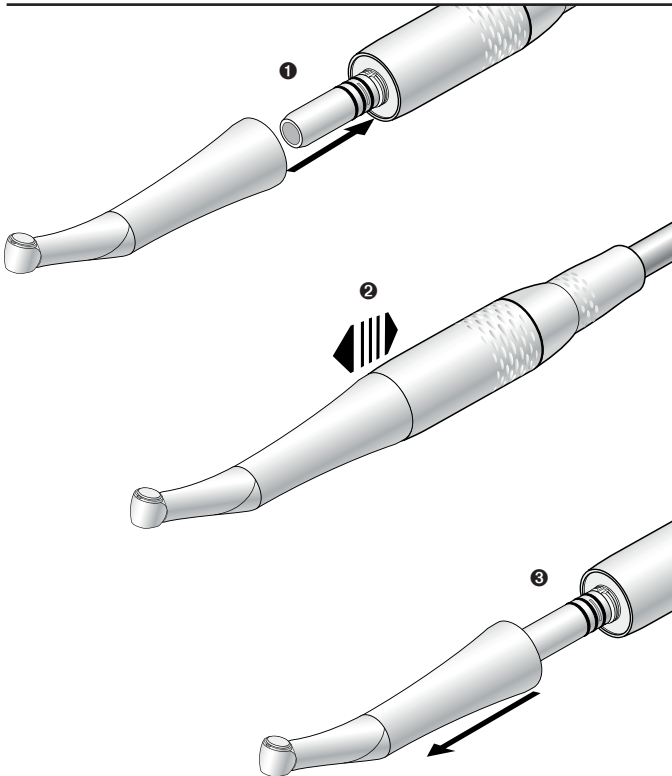
This medical device is only intended for professional use in endodontics in accordance with the applicable health and safety at work regulations, accident prevention procedures and these directions for use. It may only be reprocessed and serviced by professionals who have received instruction in infection protection, self-protection and patient protection. Improper use, e.g. lack of hygiene and device care, non-compliance with the manufacturer's instructions or use of accessories and spare parts not approved by VDW, releases VDW from any warranty or other obligations.

8.1. Getting Started

Hygiene and device care prior to first use

Upon delivery, the contra-angle has been cleaned and shrink-wrapped in PE film. Before using the contra-angle for the first time, lubricate and sterilise it (see Section 9).

Attachment and detachment

❶ Attach the contra-angle to the motor so that it snaps into place.

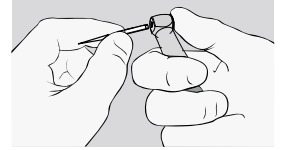
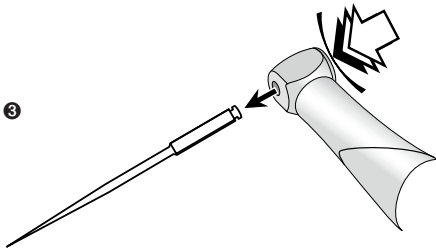
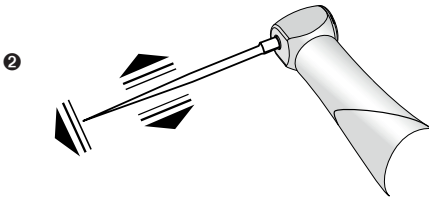
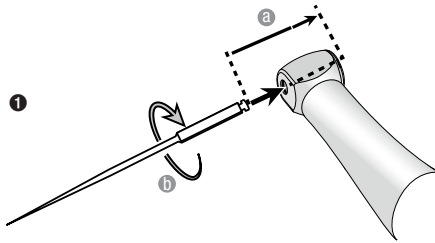
❷ Check to make sure it is securely attached to the motor.

❸ To detach, remove the contra-angle by pulling axially.

 air-motor: max. speed
e-motor: 20.000 rpm

8.2. Changing the Instrument

Only use the original RECIPROC® instruments from VDW and follow the instructions in their directions for use.

❶ Push the instrument in as far as it will go (a) and turn it until the shaft end snaps into place (b).

❷ Check to make sure it is securely attached by pulling axially.

❸ To remove the instrument, press the push button.

8.3. Clockwise/Anticlockwise Rotation

Clockwise rotation of the drive motor (standard drive movement)

The instrument operates in reciprocating mode.

Anticlockwise rotation of the drive motor

The instrument is stationary.

8.4. Trial Run

- Insert the instrument as described in 8.2.
- Start your motor and thus put the contra-angle into operation.

 air-motor: max. speed e-motor: 20.000 rpm
--

- In the event of malfunctions (e.g. vibrations, unusual noises, heating of the device) take the contra-angle out of operation immediately and contact your local VDW service partner.

9. Infection Control and Maintenance

Regular maintenance of RECIPROC® direct is essential to ensure that work performed is completely hygienic and that the device will have a long service life.

RECIPROC® direct has not been disinfected or sterilised upon delivery. The contra-angle must be cleaned, disinfected and sterilised before the first treatment and after every subsequent treatment in order to prevent soiling from accumulating. According to a recommendation by the Robert Koch Institute, a distinction must be made between reprocessing in a machine and manual reprocessing, whereby reprocessing in a machine is always preferable. In your responsibility for the sterility of the contra-angle, always ensure that only validated methods of cleaning, disinfection and sterilisation are used, that the equipment (disinfector, steriliser) is subjected to regular maintenance and inspections and that the validated parameters are maintained in every cycle.

Comply with your national guidelines, standards and requirements for cleaning, disinfection and sterilisation. Here are instructions concerning machine-based and manual reprocessing, lubrication and sterilisation. Comply with the following notes before you commence cleaning, disinfection and sterilisation.

- 
 • VDW recommends machine-based reprocessing in a washer-disinfector (WD). Please refer to the manufacturer's information concerning equipment, cleaning agents and rinsing agents.

- Always wear protective clothing and use protective gloves.
- Detach the contra-angle from the motor.
- Clean and disinfect the contra-angle immediately after every treatment in order to rinse out any liquids that have entered the device (e.g. blood, saliva, etc.) and prevent internal parts from jamming.
- Sterilise the contra-angle directly after manual or machine-based cleaning and disinfection and lubricate it before sterilisation.
- Only use cleaning agents and disinfectants that are certified by officially recognised institutes (e.g. VAH/DGHM or FDA Clearance or CE marking), contain no chlorine, have been declared aldehyde-free and have no protein-binding effect.
- Please refer to the manufacturer's information concerning use of the cleaning agent and disinfectant with regard to declared concentrations, temperatures and exposure times.

Reprocessing limitations

- Frequent reprocessing only has minimal effects on the service life of the contra-angle. The end of the product service life is dependent on wear and damage caused by use.

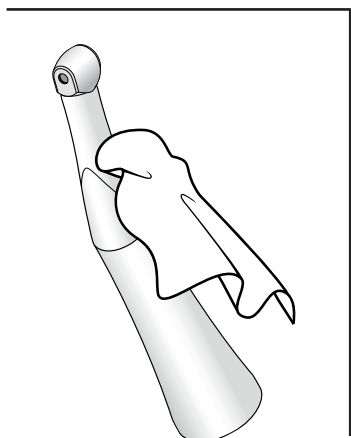
9.1. Preparation

Preparation at the place of use

- Remove the instrument and separate the contra-angle from the motor.
- Transport: No special transport container is required for the contra-angle. Transport is not subject to any special requirements.
- Wear protective gloves when you prepare and transport the contra-angle.

Manual cleaning preparation


- Manual pre-cleaning is required for an efficient cleaning procedure. Machine-based reprocessing of medical devices also begins with a manual cleaning step.
- In the event of heavy soiling, pre-clean with disinfection wipes immediately after use. Only use cleaning agents that do not have a protein-binding effect. Rinse the products under running water if necessary in order to remove any coarse soiling.
- It is not necessary to dismantle the contra-angle for this purpose.
- Do not place the contra-angle in a disinfectant solution or in an ultrasonic bath.

9.2. Manual Reprocessing

If you clean and disinfect the contra-angle manually, heed the notes below and proceed as follows:

Manual cleaning

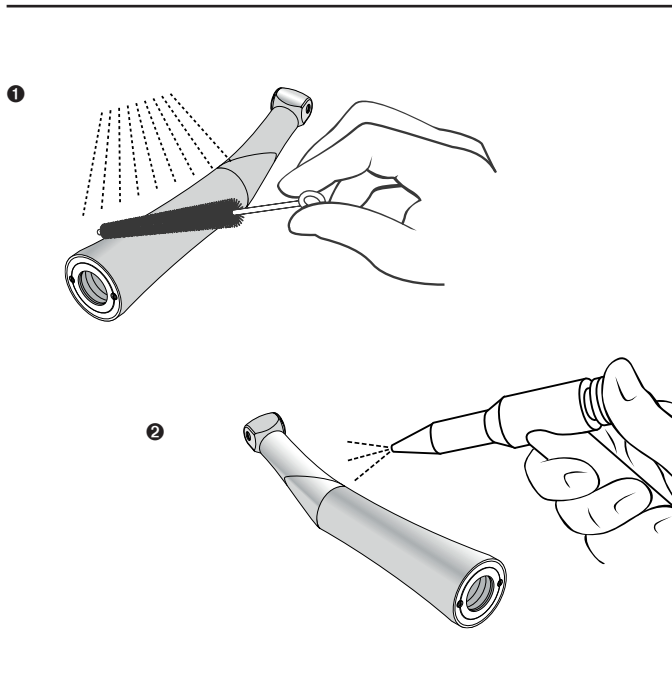Equipment: Cleaning tool (brush)

❶ For at least 5 minutes, rinse and brush under running, deionised (DI) water, or water that has this degree of purity (< 38 °C bzw. < 100 °F).

❷ Remove any liquid residues, for example using an ultra-absorbent cloth or particle-free compressed air.

Manual disinfection

- VDW recommends wipe-down disinfection with mikrozyd® AF wipes manufactured by Schülke.
- Thoroughly wipe down surfaces with a soaked wipe (for at least 30 seconds), making sure the entire surface is wetted, and allow to act (exposure time > 30 minutes). If large surfaces have to be treated, use more than one wipe where necessary.
- Make sure the surface is completely dry after disinfection.

Manual cleaning, disinfection and lubrication must be followed by sterilisation (packaged) in a Class B steam steriliser (conforming to EN 13060).

9.3. Machine-Based Reprocessing

VDW recommends machine-based cleaning and lubrication using the W&H Assistina 3x3. Follow the manufacturer's instructions in the Assistina directions for use and heed the following notes:

The RECIPROC® direct contra-angle can be cleaned and disinfected in a thermal disinfecter.

Automatic cleaning / disinfection

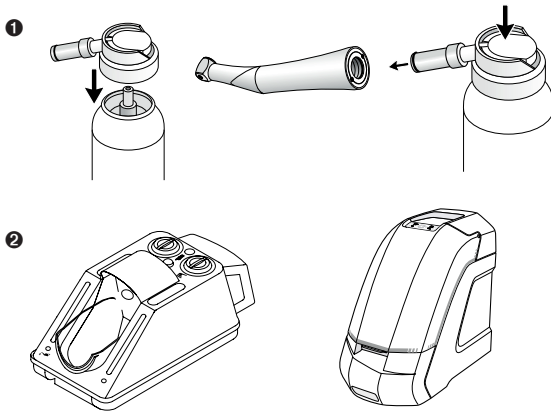
- Equipment: Washer-disinfector WD as per ISO 15883, e.g. Steelco DS 50
- Cleaning solution: e.g. Dr. Weigert neodisher mediclean forte
- Rinsing agent: e.g. Dr. Weigert neodisher mediKlar
- Pre-rinse: 2 minutes
- Cleaning at 60°C (140°F), holding time 3 minutes
- Rinse: 1 minute
- Disinfection at 90°C (194°F), holding time 1 minute
- The water quality to be recommended for cleaning is deionised (DI) water, or water that has this degree of purity.

- Please refer to the manufacturer's information concerning equipment, cleaning agents and rinsing agents.
- Drying is performed automatically during the drying cycle in the washer-disinfector. Make sure the contra-angle attachment is completely dry after thermal disinfection. Remove any liquid residues, for example by using particle-free compressed air or an ultra-absorbent cloth.
- Lubricate the dry contra-angle immediately after thermal disinfection.


9.4. Lubrication

It is essential to subject the contra-angle to device care after every internal cleaning procedure, before every sterilisation or after 30 minutes of use, but at least once a day.

- 1 For manual lubrication use the spray adaptor supplied and follow the directions for use on the oil spray can, e.g. W&H Service Oil F1 MD-400.
- 2 VDW recommends reprocessing the contra-angle using a fully automatic machine, e.g. the Assistina from W&H.

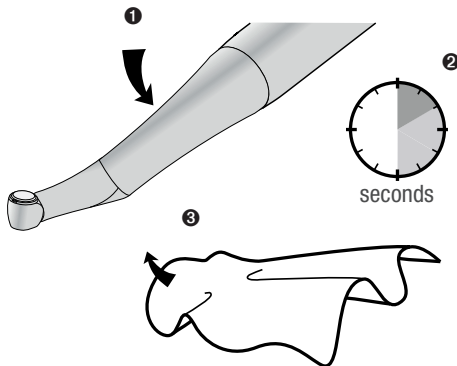Trial run after lubrication

- 1 Hold the contra-angle upside down.
- 2 Run the contra-angle for 30 seconds to remove any excess lubricant. Start at the lowest speed and after 5 – 10 seconds increase to a speed of 20000 rpm (electric motor) or to the maximum speed (air motor). If emerging lubricant contains deposits, repeat the entire infection control and maintenance process.
- 3 Wipe the contra-angle with a paper tissue or soft cloth.

9.5. Sterilisation

Packaging

- Shrink-wrap the contra-angle and accessories in sterile barrier packages conforming to EN 868-5.

Sterilisation

VDW recommends sterilisation in accordance with EN 13060, Class B.

- Please refer to the information provided by the device manufacturer.
- Clean and disinfect the contra-angle before sterilisation. Before sterilisation it should also be lubricated.
- Make sure you only remove dry, sterile items.
- Maximum sterilisation temperature 135°C (275°F).
- The sterilisation validation procedure must be performed in compliance with ISO 17665 (valid Installation Qualification and Operational Qualification (IQ and OQ) and product-specific Performance Qualification (PQ)).

Approved sterilisation procedure

- Comply with your national guidelines, standards and requirements for cleaning, disinfection and sterilisation.
- Steam sterilisation with fractionated pre-vacuum method type B, in accordance with EN 13060. The steriliser must have been explicitly approved by its manufacturer for the sterilisation of contra-angles.
- Sterilisation holding time at least 3 minutes at 134°C (273°F), at least 4 minutes at 132°C (270°F) or at least 15 minutes at 121°C (250°F).
- The fast sterilisation procedure and the sterilisation procedure with a non-packaged contra-angle are not permissible. In addition, do not use hot-air sterilisation, radiation sterilisation or sterilisation with formaldehyde, ethylene oxide or plasma.

Storage

Store sterile items in a dry, dust-free place at room temperature.

Additional notes

- Before resuming operation: wait until the contra-angle attachment has cooled down and has dried completely.
- Contra-angles must be checked for damage before they are reused. Damaged products must be replaced.
- The standard ISO 7785-2 / ISO 14457 stipulates a durability of at least 250 sterilisation cycles. We recommend subjecting the RECIPROC® direct contra-angle to regular servicing after 1000 sterilisation cycles or after one year.

10. Instruments and Accessories

Only use original VDW instruments and accessories with the manufacturer's instructions as amended (available on the VDW website (www.vdw-dental.com):

- Original RECIPROC® instruments (see current VDW catalogue)
- Oil spray adaptor for manual lubrication (REF V04 1185 000 506)

11. Technical Data

RECIPROC® direct

Motor-side coupling as per standard	ISO 3964
Recommended instruments	RECIPROC® instruments from VDW for reciprocating root canal preparation
Min. clamping length (mm)	engaging
Max. motor speed	25000 rpm
Recommended motor speed	20000 rpm
Ambient conditions	Use: in enclosed spaces Ambient temperature: 15°C / 35°C (59°F / 95°F) Relative humidity: < 80%; non-condensing at 0° (32°F) Maximum housing temperature according to EN 60601-1:2006

By selecting correct operating conditions in accordance with the manufacturer's instructions it is the user's duty to ensure that there is no risk to the user, patient or any third parties. Please always refer to the information in the directions for use for RECIPROC® instruments.

12. Service

In the event of malfunctions, please contact a local VDW service partner immediately. Repair and maintenance work may only be performed by a local VDW service partner.

13. Storage and Disposal

Conditions for transport and storage	Ambient temperature: -20°C / +50°C (-4°F / 122°F) Relative humidity: 20% - 80%, non-condensing at > 40°C (104°F)
Disposal of the contra-angle	Comply with your national laws, guidelines, standards and requirements for the disposal of the contra-angle. Make sure the parts are not contaminated when they are being disposed of.
Disposal of packaging	The packaging materials have been selected according to environmental compatibility and disposal technology aspects so they are recyclable. Any packaging materials no longer required should be handed in at the collection and recycling system. That way you help to ensure that raw materials are reclaimed and waste is avoided.

14. Exclusion of Liability

In the following cases the manufacturer declines all responsibility:

1. Use of RECIPROC® direct for purposes and applications that differ from the specific requirements made in these directions for use.
2. Cleaning, disinfection and sterilisation methods contrary to the instructions in these directions for use.
3. Contra-angle maintenance contrary to the instructions in these directions for use.

15. Warranty

This VDW product has been manufactured with the utmost care by highly qualified professionals. Many different tests and inspections ensure perfect performance and compliance with applicable laws and ordinances. Please note that warranty rights only apply if all the instructions in the enclosed directions for use have been followed.

As of the date of purchase, VDW can be held liable as a manufacturer for any material or manufacturing defects within a warranty term of 12 months.

This warranty only covers the exchange or repair of individual components or parts that are affected by manufacturing defects. The costs incurred by the dental dealer's technical personnel for visiting the customer and the customer's cost of packaging will not be reimbursed by VDW GmbH.

The customer is not entitled to file claims with VDW GmbH that go beyond repair, e.g. indemnity claims. This warranty does not cover any compensation for direct or indirect personal injury or damage to property of whatever nature. The customer is not entitled to demand compensation for device outage times.

The warranty does not cover damage or losses where VDW GmbH proves that they occurred as a result of any omissions by the user concerning normal device care (as referred to in the directions for use). The warranty explicitly excludes defects that:

- have been caused by damage in transit to VDW GmbH / to a service partner authorised by VDW for the purpose of repair,
- have been caused by atmospheric events such as lightning strike, fire and moisture. This warranty is automatically rendered void if the product has been improperly repaired, modified or otherwise tampered with by the user or unauthorised third parties.

Legal claims, resulting from product-liability law for example, and claims filed with the person from whom the customer purchased the product, especially ones filed with the dental dealer, shall remain unaffected.

Any warranty claims must be filed, enclosing proof of purchase, with the supplier or a local VDW service partner. The rendering of a warranty service extends neither the term of the warranty nor the term of any statutory guarantee.

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VDW GmbH • Bayerwaldstr. 15 • 81737 Munich • Germany